

FreshKon®

FREQUENT REPLACEMENT COSMETIC CONTACT LENSES

FreshKon® Alluring Eyes • FreshKon® Mosaic
FreshKon® Colors Fusion • FreshKon® Designer

A Guide to Trouble-Free Contact Lens Wear (Instructions for Use)

Consult an Eye Care Professional for suitability of wear

www.freshkon.com

Contents

Two sterile daily wear soft contact lenses, 45% methafilcon A, 55% water, immersed in 0.9% buffered saline.

Indications for Use (Intended Purpose)

Indicated either for emmetropia or for the correction of refractive ametropia in aphakic or non-aphakic persons with non-diseased eyes. It is also meant for cosmetic purpose.

Lens Insertion


1. Wash your hands thoroughly with soap and dry them with a lint free towel. Rinse the lens thoroughly with FreshKon[®] Multi-Purpose Solution.
2. Place the lens on the tip of your index finger. Make sure the lens is in a bowl shape and correctly oriented.



3. Hold up your upper eyelid firmly with one finger of your other hand.



4. While holding down the lower eyelid with one finger of your applying hand, carefully place the contact lens on your cornea.

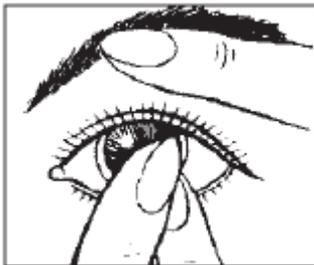

5. Slowly release upper eyelid and gently close your eyes.



6. Repeat the same procedure for the other eye.

Lens Removal

1. Wash your hands thoroughly with soap and dry them with a lint free towel.
2. Check to ensure lenses are on your cornea.
3. Hold down lower eyelid with your middle finger.
4. Use forefinger to slide lens to lower part of eye.
5. Gently pinch the bottom edge of the lens with the pads of your index finger and thumb to remove lens.



6. Repeat the same procedure for the other eye.


Daily Cleaning and Care Regimen

1. After removing the lens from the eye, place it on the palm of your hand and apply a few drops of FreshKon[®] Multi-Purpose Solution onto the lens.
2. Rub the lens gently with your index finger against the palm for about 20 seconds on both faces.
3. Rinse the lens thoroughly with FreshKon[®] Multi-Purpose Solution.
4. Soak the clean lens in the case with FreshKon[®] Multi-Purpose Solution for your next use.
5. Never rinse your lenses in water from the tap.

6. Do not change the contact lens care system without consulting an Eye Care Professional.

Schedule for Daily Wear/ Recommended Replacement Schedule/ Follow-up Visits


Schedule for Daily Wear

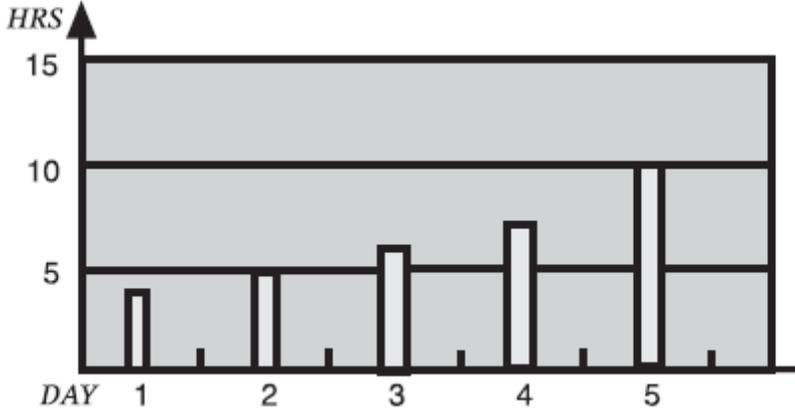

Note: You may notice a slight blur to the side when first wearing cosmetic contact lenses. This is normal for cosmetic contact lenses. Vision will improve after the wearer undergoes an adaptation period.


1. Never wear your lenses beyond the period recommended by your Eye Care Professional
2. Always discard lenses worn on a frequent replacement schedule after the recommended wearing schedule prescribed by your Eye Care Professional.
3. The lenses are meant for daily wear and monthly disposable.

Follow-up visit

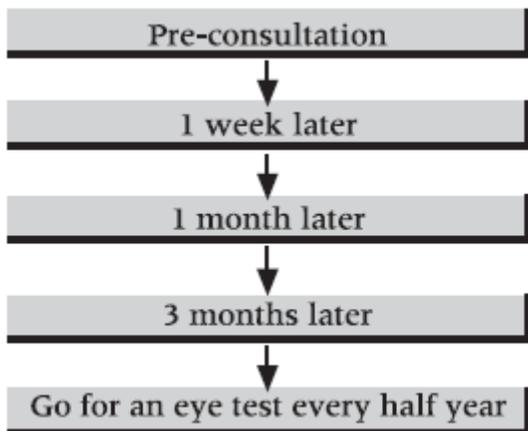

Contraindications/ Warnings/ Precautions/ Adverse Reactions

1. Do not use the contact lens if any of the following conditions exist: (a) Any eye disease, injury, or abnormality that affects the cornea, conjunctiva, or eyelids (b) Any active corneal infection (bacterial, fungal, protozoal, or viral) (c) If eyes becomes red or irritated.
2. Proper use and care of contact lenses and lens care products, including lens cases, are essential for the safe use of these products.
3. If you experience eye discomfort, excessive tearing, vision changes, redness of the eye or other problems, you should immediately remove your lenses and promptly contact your Eye Care Professional. It is recommended that you see your Eye Care Professional routinely, as directed.
4. Do not use if the sterile package is opened or damaged.
5. Always keep your lenses immersed in recommended storage solution when lenses are not being worn (stored). Prolonged periods of drying will reduce the ability of the lens surface to return to a wettable state.
6. Avoid all harmful or irritating vapors and fumes while wearing lenses.
7. Ask your Eye Care Professional about wearing lenses during sporting activities.
8. Put on your lenses before putting on makeup. Do not get cosmetics, lotions, soaps, creams, deodorants, or sprays in your eyes or on your lenses.
9. Do not share your contact lenses with others.

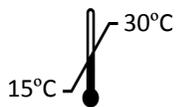

Printed in Singapore
Revision date: 3/2012
Revision number: MM-IFU(4)



Obelis s.a
Boulevard General Wahis 53
1030 Brussels, BELGIUM



Oculus Private Limited
61 Tai Seng Avenue,
#05-12,
Singapore 534167